

ENT BOTTLE CLEANING INSTRUCTIONS

GE8060 • GE6030 • GE7050 • GE7070 • GE8000

- To Clean** Disassemble completely and place in ultrasonic cleaner for 10 minutes.
- To Disinfect** Use approved disinfectant solution by direction of manufacturer of solution.
- To Sterilize** Use approved sterilization solution by direction of manufacturer of solution.
- To Dry** Use the compressor from cabinet to dry completely prior to refilling.
- To Autoclave.....** All pieces and parts may be standard steam autoclaved by direction of manufacturer of autoclave only after using an ultrasonic cleaner.
(Do Not Flash sterilize).

***Note: Autoclaving will cause discoloring of parts
and will shorten the life of all o-rings.***



7740 Records St. • Indianapolis, IN 46226
800-428-1610 • (317) 545-6196 • Fax (317) 543-3289
www.anthonypproducts.com