


Now You Can
Breathe Clearly
After Any Nasal or Sinus Surgery!



RELTOK CLEAR-FLO™
NASAL AIRWAY SYSTEM

- No more blocked nasal passages after surgery
- Aids in preventing ear popping, claustrophobia, & anxiety during recovery
- Simple, quick, pain-free removal
- Latex-free, FDA-cleared medical grade silicone

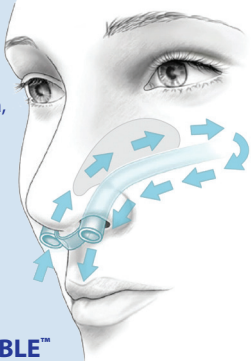

**ALWAYS CLEAR,
ALWAYS COMFORTABLE™**

Introducing the **Reltok Clear-Flo™** Nasal Airway



This new medical device, a pair of soft silicone tubes, is inserted by a nasal surgeon at the end of any nose or sinus operation. It allows patients to breathe immediately after surgery, and throughout the first one to five day post-operative period.


Whether the nose has been packed or not, without being able to breathe through the nose, patients have been miserable and not enthusiastic about the operations. One unhappy patient likened the experience to "having a clothespin on my nose for five days".

The Reltok Clear-Flo™ Nasal Airway is the patient's best friend after surgery because it provides clear nasal passages and erases patient anxiety, claustrophobia as well as clogged ears and dry throat which detract from the satisfaction of a successful surgery.

This device and accessories establish a new standard of care for any nasal or sinus operation since patient comfort becomes a given.

An End to Block Nasal Passages!


Because we care about your comfort and safety, during your nasal/sinus surgery we insert the new **RELTOK CLEAR-FLO™ NASAL AIRWAY** because we want you to breathe clearly and comfortably, beginning immediately after surgery.

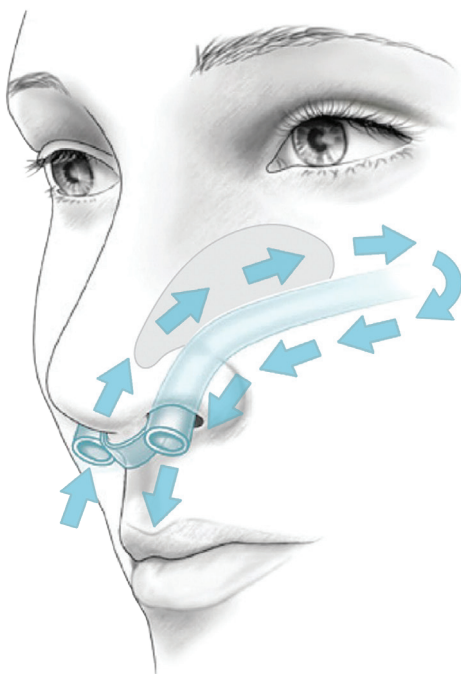

Reltok Clear-Flo® Nasal Airway Benefits:

- Allows air to pass as it normally does through the nose en route to the lungs
- Aids in preventing blocked nasal passages, ear popping, claustrophobia, and anxiety during recovery
- Well-tolerated for up to 10 days
- Made from FDA-cleared Class VI medical grade silicone
- Non-stick, latex-free
- The tubes sit on the floor of the nasal passages, hence, there is no pain; you will not feel their presence
- Compatible with all nasal/sinus packing materials
- Topical anesthetic administered into nose makes removal in office, simple, quick and painless
- 98% patient satisfaction rating

Post-Operative Care

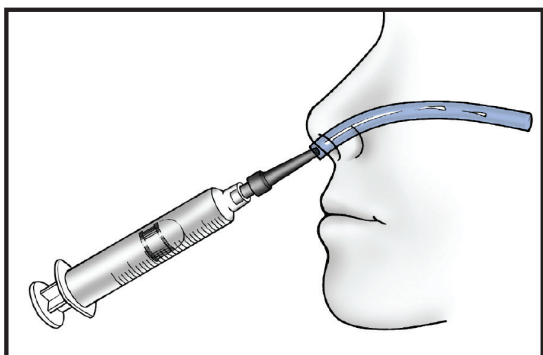


Your surgeon has placed a double-tube airway inside your nasal passages. This device allows normal breathing after nasal surgery, even if your nose has been “packed”.

To keep the two airway tubes open, you will take home a small syringe and plastic tip. You, or your caretaker, can easily and painlessly flush the tubes to clear any blood or mucus that may accumulate.

Your recovery room nurse will attach the tip to the syringe and demonstrate, to you and your caretaker, how to flush the tubes.

If you have any questions, contact your surgeon.

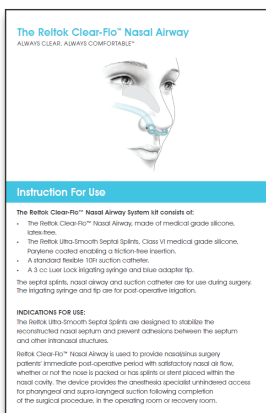

*Easy-to-follow Illustrated
Home Care Instruction Sheet*

Quick, Painless Removal

The airway will be removed by your surgeon, in their office, in accordance with their particular post-operative timetable.

The device is made of soft, latex-free, medical-grade silicone that does not stick to the inside of the nose. Therefore, removal is quick and virtually painless.

What Patients Are Saying About the New Reltok Clear-Flo™ Nasal Airway

The device's most ardent champions have been patients whose previous nasal operation **did not** have the airway and the subsequent operation did.

These patients, veterans of both varieties of recovery, offer the most powerful testimonials in behalf of the Reltok Clear-Flo™ Nasal Airway System.

"The first time I had surgery, the packing caused the most discomfort during the recovery process. The Reltok Clear-Flo Nasal Airway was really easy to take care of and whenever I felt it getting clogged, I could flush it out and feel better. I couldn't feel the AIRWAY in my nose and it wasn't uncomfortable. I also felt it was easier to keep clean."

– Michelle Ocompo

"The most amazing thing was that I could breathe after surgery!!! I had never experienced this with my other surgeries nor had I ever heard of it. The fact that I could breathe out of my nose because of these tiny tubes made recovery so much more comfortable and easy. All I can say is "WOW!"

– Georgia Mayfield

"Having the Reltok Clear-Flo Nasal Airway was much better. It allowed me to breathe through my nose, which made the whole experience less painful and more comfortable."

– Inga Grigorian

"The Reltok Clear-Flo Nasal Airway was my "life line" in being able to breathe and relax. I cannot imagine doing this procedure without it."

– Evelyn Valenzuela


Read more patient testimonials
& additional information at
www.reltok.com

1. Select "For Patients" from the top navigation menu bar.
2. Next browse the "For Patients" tabs located on the left side of the screen.



RELTOK CLEAR-FLO™

NASAL AIRWAY SYSTEM

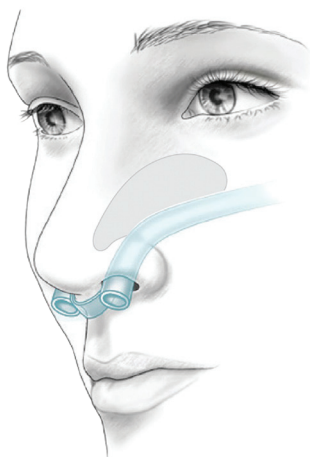

**ALWAYS CLEAR,
ALWAYS COMFORTABLE™**

***Ask us about this new device
that allows clear breathing
after your nasal surgery!***

The Reltok Clear-Flo™ Nasal Airway was developed by
Robert Kotler, MD, FACS, Clinical Instructor, Head and Neck Surgery, UCLA.
Clear-Flo Nasal Airway™, Always Clear, and Always Comfortable are trademarks of Reltok Nasal Products.
©2021 Reltok Nasal Products, LLC U.S. Patent No. 8,092,478 and 8,874,486.

Proudly made in the USA.