

AGNI-3 NASAL IRRIGATOR



API's AGNI-3 Irrigator is made of delrin and stainless steel. The AGNI-3 is reusable and may be autoclaved or boiled in order to sterilize it.

api
anthony products, inc.

AGNI-3 DIRECTIONS FOR USE

1. To be prescribed by a physician **ONLY**. Be sure physician has explained all instructions clearly before using.
2. Attach nasal irrigator to the pulsating water irrigation unit.
3. Add 1tsp. of salt to the unit basin and fill with warm water.
4. Always set unit at lowest level of pressure.
5. Holding finger grip, place delrin end into one nostril and lean over sink.
6. Turn machine on.
7. Saline solution enters nostril, bathes the nose and flows out the other nostril. Use unit until the fluid is clear.
8. Rinse machine with clear water to remove salt.
9. Discontinue use when nasal condition is cleared or as directed by your physician.