

AGTI-3 THROAT IRRIGATOR



API's AGTI-3 Irrigator is made of delrin and stainless steel. The AGTI-3 is reusable and may be autoclaved or boiled in order to sterilize it.

api
anthony products, inc.

AGTI-3 DIRECTIONS FOR USE

- 1.** To be prescribed by a physician **ONLY**. Be sure physician has explained all instructions clearly before using.
- 2.** Attach throat irrigator to the pulsating water irrigation unit.
- 3.** Add 1tsp. of salt to the unit basin and fill with warm water.
- 4.** Always set unit at lowest level of pressure.
- 5.** Holding finger grip, place delrin end into mouth and lean over sink.
- 6.** Turn machine on.
- 7.** Saline solution enters mouth and bathes the throat. When finished, spit saline solution out into sink.
- 8.** Rinse machine with clear water to remove salt.
- 9.** Discontinue use when throat condition is cleared or as directed by your physician.