

DolphinPods PLUS

INSTRUCTIONS FOR USE

MAKING THE SOLUTION:

REMOVE Spray Head (Turn Counterclockwise)

FILL Half Way with Water

DROP 1 DolphinPod in Water

FINISH filling the bottle with water

RE-ATTACH Spray head after 2 minutes

TO PRECLEAN SURFACE:

1. Spray DolphinPods Plus solution directly on surface or towelette and wipe area to preclean of all visible debris.

DISINFECT AFTER CLEANING:

1. Spray DolphinPod Plus solution directly onto surface, thoroughly wetting area. Leave surface wet for 5 minutes.*
2. Wipe with clean towelettes. Once complete, dispose of used towelette. Dispose of used towelettes in accordance with Federal, State, and local regulations for infectious materials disposal.

* On non-soiled surfaces, DolphinPods Plus have been tested to kill many organisms in ≤ 1 minute, including MRSA, Pseudomonas, and E. Coli

- Refer to chair manufacturer instructions for use on chairs. Barrier technique and/or soap and water are recommended for upholstery. It is possible for discoloration to occur on upholstery. A spot check and precleaning are recommended if the DolphinPod solution is to be used on chairs and soft plastics.
- Use of DolphinPods Plus on brass, copper, anodized aluminum, and soft plastic; could discolor/or damage surfaces, if not used as directed and over applied.
- Use of gloves with corn starch may cause yellowing of surfaces.
- CDC and Dental Association guidelines recommend the use of utility gloves when cleaning and disinfecting surfaces.