

Cleaning Instructions

for Anthony Products

ENT Bottles



In Box
GE6030



In Box
GE8060



In Box
GE5010



In Box
GE8000



In Box
GE7050



In Box
GE7070

Distributed by

api

anthony products, inc.

Manufactured for CAT Products, LLC

To Clean

Disassemble all parts and put in ultrasonic cleaner. Blow all parts dry with compressor.

To Sterilize

Use approved liquid sterilant or disinfectant solutions.

Do not autoclave.

Distributed by

Anthony Products, Inc.

7740 Records Street • Indianapolis, IN 46226

(317) 545-6196 • 800-428-1610

www.anthonyproducts.com

LGEIN04 03 10