

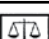
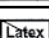
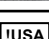
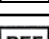
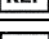
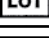



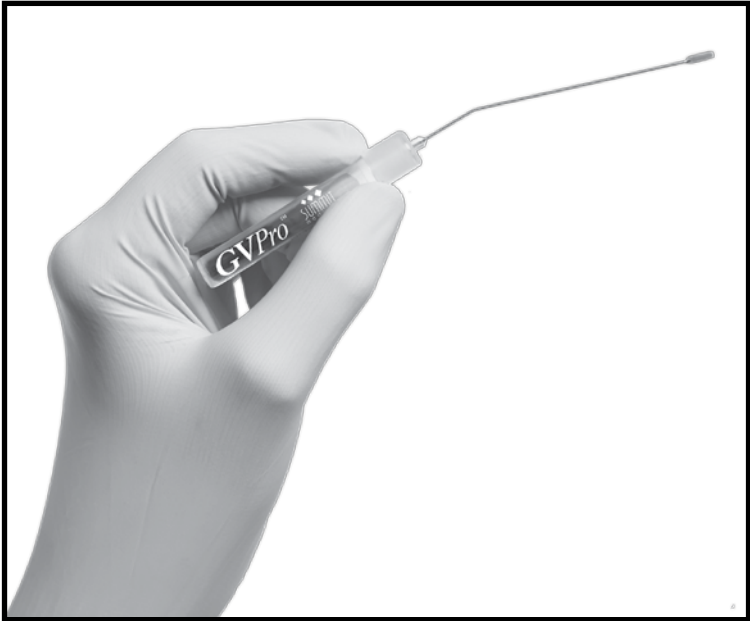


# GVP<sup>Pro</sup><sup>TM</sup>

## Gentian Violet Applicator

### Symbol Reference Key

	Consult instructions for use
	Non-Sterile
	Quantity
	Latex Free
	For US audiences only
	Catalog Number
	Lot Number
	Use By
	Date of Manufacture
	Keep away from sunlight
	Do Not Reuse



## Instructions for Use

**NON-STERILE**




Single-use, one-patient device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness or death of the patient.

Distributed by



anthony products, inc.

 GVP<sup>Pro</sup>™ Gentian Violet Applicator is manufactured by and is a trademark of Summit Medical, Inc. All rights reserved. Patent pending.

Distributed by

**Anthony Products, Inc.**

7740 Records Street • Indianapolis, IN 46226

800-428-1610 • (317) 545-6196 • Fax (317) 543-3289

[www.anthonyproducts.com](http://www.anthonyproducts.com)

# GVP<sup>Pro</sup>™ Gentian Violet Applicator

Before using, read the following information:

**NON-STERILE**

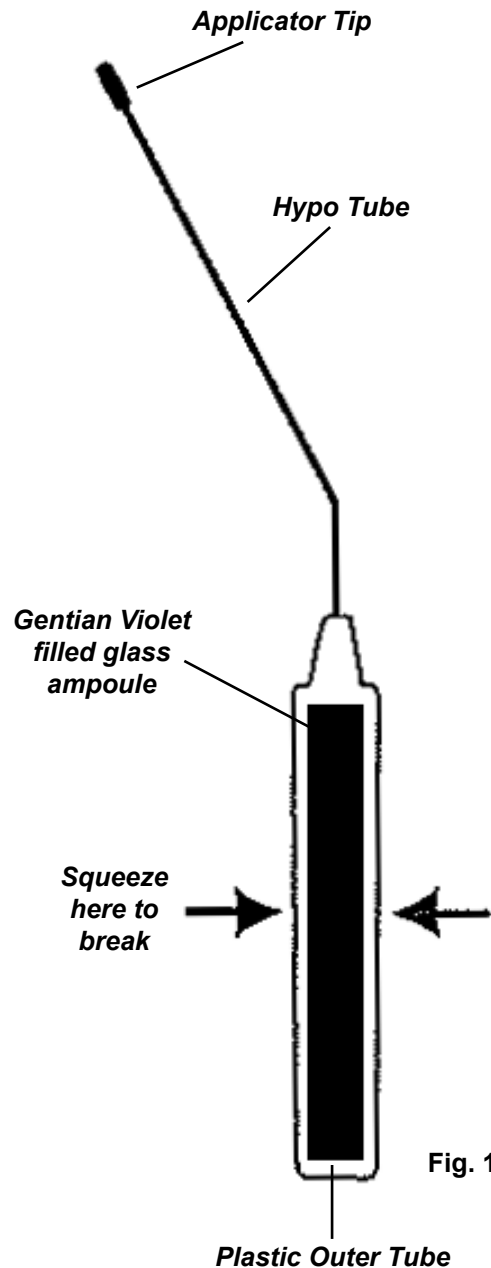


Fig. 1

## CONTENTS

10 individually wrapped applicators, containing 0.70ml gentian violet solution (1% gentian violet, 10% ethanol, purified water).

## FOR PROFESSIONAL USE ONLY

## STORAGE

Store at room temperature. Protect from light and heat.

## INSTRUCTIONS FOR USE

The ampoule is closed and the gentian violet solution is not in contact with the applicator.

1. Wear gloves when handling.
2. Open the pouch and remove the applicator.
3. Hold upright (applicator tip up, ampoule down) and squeeze the plastic tube by applying pressure to the middle of the ampoule inside the tube (Fig. 1). This breaks the ampoule releasing gentian violet into the plastic tube.
4. Turn the applicator over (applicator tip down, ampoule up) to start the gentian violet solution flowing towards the applicator tip.
5. The gentian violet solution is delivered into the hypo tube and onto the applicator tip by squeezing the plastic tube.
6. Place the applicator tip near the desired application site prior to squeezing the plastic tube.
7. The applicator is ready to use when the applicator tip darkens with gentian violet solution.
8. After use, place applicator back into foil pouch and dispose of in medical waste.

## WARNINGS

- For external use only.
- Do not use in the eyes.
- Keep out of reach of children.
- Tattooing of the skin may occur if applied to an ulcerative lesion.
- In case of accidental ingestion, contact a Poison Control Center immediately.
- Keep away from heat.
- Keep out of sunlight.

## DISPOSAL

Treat all used GVP Pro applicators as medical waste. Dispose of according to local laws and regulations for such waste.