

# Insta-Hold Instrument Holding Drape

**Directions For Use:** Remove drape from package. Peel off clear plastic liner from tacky surface. Pre-wash mat with warm water soap solution. Rinse well and pat dry with lint free absorbent material.

Sterilize and position mate over surgically draped patient with tacky surface face up.

**Sterilization:** Place lint free cloth or paper towel over tacky surface and loosely fold or roll.

The drape can be flashed, steamed autoclaved or gas sterilized using standard sterilization temperatures and procedures.

**To Reclean:** Use a soft textured sponge, gently wash tacky surface with warm water soap solution. Rinse thoroughly and pat dry with lint free absorbent material.

The above recleaning and sterilization instructions are only general guidelines; standard and approved hospital procedures for handling a reusable product should be followed.

**NOTE: TO PRESERVE SURFACE HOLDING POWER, DO NOT WIPE OR RUB THE TACKY SURFACE WITH ANY ABRASIVE DRY CLOTH, SPONGE OR TOWEL. INSTA-HOLD'S HOLDING CAPABILITIES MAY BE REDUCED BY LINT, POWDER, DRIED BLOOD OR FATTY TISSUE, BUT CAN BE IMMEDIATELY RESTORED BY RINSING OR GENTLY WASHING WITH A WET SPONGE.**

# Insta-Hold Instrument Holding Drape



Non-magnetic reusable instrument holding mat non-sterile

- Size: 19" x 12"**
- Size: 19" x 24"**

7740 Records St. • Indianapolis, IN 46226  
800 428-1610 (317) 545-6196 • Fax (317) 543-3289  
[www.anthonyproducts.com](http://www.anthonyproducts.com)

**api**  
anthony products, inc.